

PLURAFOL

Gebrauchsanweisung (deutsch/Rev. 03/20)

Für Dampf- und Gassterilisatoren gemäß EN 868-5 und ISO 11607-1

Anwendungshinweise

Die Sterilisationsfolie Plurafol dient als Verpackung für medizinische Produkte, die dampf- oder gassterilisiert werden sollen.

Versiegeln Sie das offene Ende mit einem Durchlauf- oder Impulsiegelgerät bei empfohlener Temperatur (170–180 °C) und Druck (70–90 N/cm²) und befolgen Sie bitte die Gebrauchsanweisung des Geräteherstellers. Die Verpackung darf nur einmal benutzt werden. Keine beschädigte Packung verwenden.

Externe Indikatoren

Plurafol

EO Gas: gelb/braun sterilisiert

Dampf: braun sterilisiert

Lagerbedingungen

Vermeiden Sie direktes Sonnenlicht und zu große Hitze.

Empfohlen werden folgende Werte:

Temperatur: 4–35 °C

Luftfeuchtigkeit: 30–70 %

Die Transportkartons sollten bei Transport und Lagerung nicht beschädigt werden.

Lieferformen

- | | |
|-------------------------------------|-------------------------------------|
| • 50 mm x 200 m
Art.-Nr. 236167 | • 150 mm x 200 m
Art.-Nr. 236170 |
| • 75 mm x 200 m
Art.-Nr. 236168 | • 200 mm x 200 m
Art.-Nr. 236171 |
| • 100 mm x 200 m
Art.-Nr. 236169 | • 250 mm x 200 m
Art.-Nr. 236172 |

Instructions for use (english/Rev. 03/20)

For stream- and gas sterilisation acc. to EN868-5 and ISO 11607-1

Use Plurafol reels to pack medical devices to be sterilized by steam or EO gas.

Seal the open end(s) with a rotary or impulse sealing machine at the recommended temperature and pressure (170–180 °C/338–356 °F and 70–90 N/cm²) and following the instructions of the sealing machine manufacturer. The packaging is for single use only. Do not use if pack is damaged.

External Indicators printed on the Products

Plurafol

EO gas: yellow/brown sterilized

Steam: brown sterilized

Storage Conditions

Keep away for direct sunlight and heat.

Store at temperature 4–35 °C (40–95 °F)

Relative humidity: 30–70 %

Outer cartons should not suffer physical damage in handling and storage.

Delivery forms

- | | |
|-------------------------------------|-------------------------------------|
| • 50 mm x 200 m
Art.-Nr. 236167 | • 150 mm x 200 m
Art.-Nr. 236170 |
| • 75 mm x 200 m
Art.-Nr. 236168 | • 200 mm x 200 m
Art.-Nr. 236171 |
| • 100 mm x 200 m
Art.-Nr. 236169 | • 250 mm x 200 m
Art.-Nr. 236172 |

